

PROSPERA...ADVANCED NEGATIVE PRESSURE WOUND THERAPY


The Prospera PRO series of negative pressure wound therapy (NPWT) systems are designed to advance the art and science of NPWT for clinicians and patients. Manufactured by DeRoyal...the PRO-II is engineered specifically for use with NPWT and provides ease of use and flexibility for clinicians, cost effective outcomes and perhaps most importantly it is designed to put the needs of patients first.

The portable PRO-II delivers all the benefits of NPWT without the pain and discomfort for the patient. Lightweight and discreet, the PRO-II is comfortable so your ambulatory and homecare patients can get on with their lives.

Prospera Advanced NPWT Solutions...combine lower than traditional negative pressure settings with the Variable Pressure Therapy, and the most suitable dressing for NPWT today – moist AMD gauze.

Benefits

- Flexible, easy to use settings with Continuous (CPT) or Variable (VPT) Pressure Therapies...lower pressure settings enhance patient comfort
- Proprietary internal and external filter system for an extra layer of safety
- Multiple alarm and safety features that protect the patient
- Variety of dressing options available...foam and gauze
- Moist AMD gauze that will not adhere to the wound substantially decreasing patient pain
- Large, easy-to-read, color-coded display screens
- Over 24 hour battery run time
- Virtually silent operation

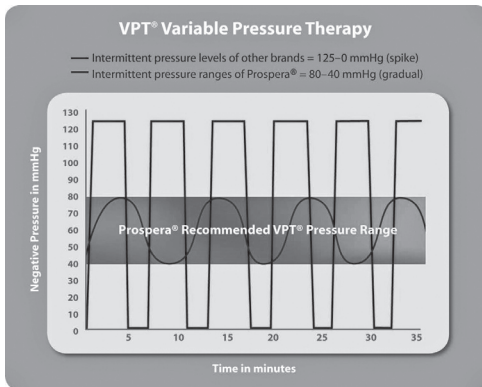

Variable Pressure Therapy (VPT)

In VPT, two different negative pressures are used, but negative pressure is always maintained, never dropped to 0 mmHg. VPT offers the advantages of intermittent NPWT, while avoiding potential for displacement of the dressing and loss of an airtight seal associated with pressure drops to 0 mmHg.

Lower Pressure

Prospera recommends negative pressure levels in the range of -80 mmHg, lower than the commonly recommended -125 mmHg. Lower pressures have been shown to reduce pain, while offering comparable effects on blood flow, wound contraction and volume reduction. Lower pressures also reduce the risk of tissue ischemia at the wound edge.

BLACKBURN'S...Regional Distributor

DeRoyal selected BLACKBURN'S as their regional provider for the Prospera Pro-II NPWT solutions. BLACKBURN'S goal is to deliver clinical-focused advanced wound care through innovative products that benefit the patients, clinicians and healthcare facilities.

Looking for a better NPWT option for your homecare and ambulatory patient? Schedule an in-service or to learn more about the Prospera PRO-II, contact a location near you.

Contact a Location Near You



Accredited Since 1995

Erie Region Pennsylvania

308 East Sixth Street
Erie, PA 16507
800-328-9325 ext. 2612
814-454-2706 (fax)

Other Regions in Pennsylvania

301 Corbet Street
Tarentum, PA 15084
800-472-2440 ext. 1511
724-224-9450 (fax)

Serving Regions of Pennsylvania, Ohio, New York and West Virginia

www.blackburnsmed.com